




Neo GBR kit

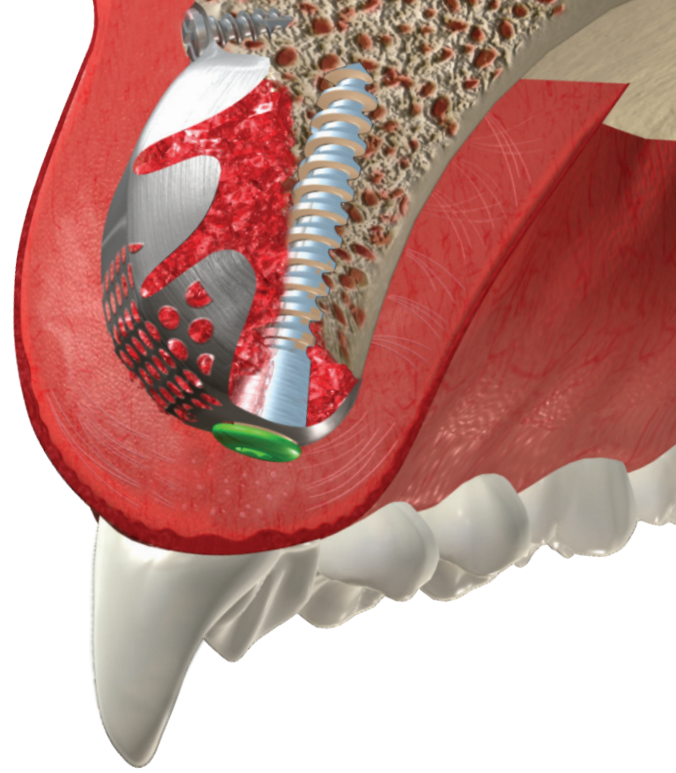

For successful GBR procedure **Neo GBR kit** (GBR total solution)



Neo GBR kit



What to do with Neo GBR kit

With Tent Screw, primary vertical and horizontal formation is possible without placing the implant
By utilizing the Cover Screw, fixation of membrane to the Tent Screw is done with ease
Secondary stabilization of the membrane is completed with Fixing Screw
With Tent Screw space for bone formation on the vast defect area can be maintained
Bone formation utilizing Spacer, Membrane and Cover Screw after placing Implant

Spacer



After connecting to the fixture, mount CTi-mem on Spacer and have it stationed using Cover Screw.



Tent Screw

Mount CTi-mem on a stationed Tent Screw and fasten using Cover Screw



Fixing Screw

After bending or customizing, the Cti-mem, tighten the Fixing screw into the Foot Fixing hole

Vertical Augmentation with Neo GBR kit

By utilizing Neo GBR kit, Tent Screw, Cover Screw and Fixing Screw, vertical and horizontal bone level can be augmented.

