



Auto Chip Maker

Maximum autogenous bone volume

Minimum time (1cc within 10 seconds)



ACM features

1. Enough high quality Autogenous bone for good bone formation, easy to apply for clinical trials
2. Collect the 1cc bone chip within 10 seconds
3. Bone chip form makes easy handling while bone grafting.
4. The bone volume can be adjusted by different diameters.
5. No slippery due to the pointer.
6. Excellent cutting force, no shaking at any surface
7. Enough irrigation through holes prevents bone heating

How to Use



Drill at 300~500rpm with enough irrigation after install the stopper
**80rpm without irrigation to harvest fresh bone*



Collect autogenous bone with a depth of 4mm



Remove the stopper gently and remove the bone to the bowl



Put the bone in the bowl



Use Autogenous bone for the bone grafting



Complete bone grafting